



Disposable Skin Staplers & Accessories

(Version No.: 20250804DS-A/8)

Model: OBS-MW & OBS-MH & OBS-MR

Indications For Use:

Disposable Skin Staplers & Accessories is a disposable sterilized equipment which applies to skin suture during general surgical operation, plastic surgery, neurosurgery operation, gynecologic surgery, burn surgery, emergency operation and field rescue and so on.


Contraindications:

- When it is not possible to maintain at least 5mm distance from the staple skin to underlying structures, the use of staplers for skin closure is contraindicated.
- The use of this skin stapler is contraindicated in patients with known sensitivities or allergies to the metals contained in 316L stainless steel, i.e., chromium, nickel, copper, cobalt (<0.01%), and iron.
- Not to be used for internal tissue closure.

Cautions and Warnings:

- These devices are restricted to sale by or on the order of a physician.
- These devices are sterilized products for single use. It is prohibit to reuse, reprocess or resterilize. Reuse, reprocessing, or re-sterilization may compromise the structural integrity of the device and lead to device failure and cause patient injury. Besides, it may create a risk of contamination and/or cause patient infection or cross infection.
- Discard of these devices should be according to the relevant local regulations.
- These devices are sterilized by Ethylene Oxide and valid for 3 years. Prohibit to use these devices when it is expired and the package is damaged or opened.
- To check the products still being expiry date before use, and check its package have no damage.
- Push a staple to the air to ensure come out smoothly and structured regularly

Instructions for Use of Disposable Skin Staplers & Accessories

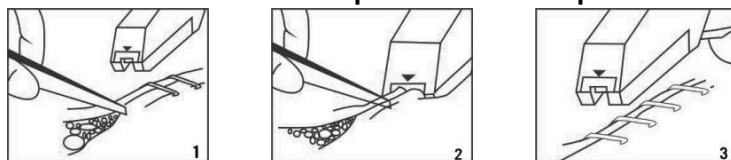

1. Use tissue forceps to pull skin edges together until edges evert.
2. Position the device over the everted skin edges, aligning the device arrow with the center of the incision. Squeeze the trigger until the handle is closed.
3. Then release the trigger and remove the device from the fired staple.

Description for model

Model: specifications , it is detailed information as follow:

- The material of staple should be used the stainless steel stipulated in ISO5832
- Every part of stapler should move smoothly. It is prohibited to get stuck or loose.
- After suturing, the staples can be removed smoothly by staple remover.
- The tip of staple should be sharp enough and insert tissue smoothly.
- The package of stapler should be secured enough. No break and open.
- staples removed strength should not over 3 kg for removing staples
- weight: about 80g

Codes	Specification		Sterilization method
OBS-MR OBS-MW OBS-MH	R=5.4X3.6mm, M=15/25/35 staples W=7.0X4.0mm, M=15/25/35 staples H=7.0X4.5mm M=15/25/35 staples	Stapler Case: Stainless steel Staples : Stainless steel Adhesive: Aron Alpha Anvil: Stainless steel Spring : Stainless steel Cartridge: ABS Pusher : Polyacetal resin	Sterilization by EO (ethylene oxide) and for single use

Accessories: Disposable staple remover

How to use: insert the remover's under two tips into the gap between skin and staple, then the third tip press the outer of the staple and strength slowly until the shape of "B" open gradually, so remove the staple depart from the body automatically.



Component	Material	Specification
handle	ABS	Medical grade
tip	304Stainless steel	0Cr18Ni9

Warranty:

These products are guaranteed against material and workmanship. The warranty is null and void should damage occur as a result of improper handling use. Care must be taken in the use and reprocessing of these products

Sterility guaranteed unless package has been opened, broken or damaged.

Disposal methods are according to local laws and regulations.

Recommended Storage conditions: the packaged product shall be stored in a dry, clean and well-ventilated room without aggressive gas and the storage temperature shall be lower than 40°C.

Package: 10pcs/box of Disposable skin staplers; 50pcs/box of Disposable staple remover

Made in China

 **Baisheng Medical Co.,Ltd.**
No.11 Fusheng Road,Xinhui District,Jiangmen,
Guangdong,China
Tel:0086-750-6628113
Fax:0086-750-6616122
www.obs-medical.com



Prolinx GmbH
Brehmstr. 56, 40239 Duesseldorf
Germany

Explanation for used symbols

	Refer to Instruction for use
	Symbol for caution
	No latex be used

	Do not reuse
	Be sure not to use if the package is damaged
	Sterile Barrier System sterilized by Ethylene Oxide
	CE marking
	Symbol for “USE BY”
	Symbol for “BATCH CODE”
	Symbol for “date of manufacture”
	Keep away from sunlight
	Keep dry
	Do not resterilize
	Symbol for “temperature limitation”
OBS	Trade mark
	Medical device
	Do not mix with the unsorted household waste.
	Contains hazardous substances